

Sample Collection Instructions

Please read the instructions completely before collecting a sample. After reading all of the instructions, if you still have questions or need technical assistance please contact your doctor.

See also the Hemocult ICT Physician Instructions.

For complete product information see the Hemocult ICT Product Instructions, included with Hemocult ICT Tests (Product No. 395067) or online at <http://www.beckmancoulter.com/rapids>



DO NOT take test:

- **three days before, during or three days after menstrual period.**
- **if you have bleeding hemorrhoids, blood in urine, open cuts on hands, or have strained during bowel movement.**

DO NOT collect sample from stool under water.

DO NOT use past printed expiration date on Collection Card (🗑️).

Store Collection Cards at room temperature 59-86°F (15°C to 30°C).

PRODUCT INFORMATION

Hemocult ICT is CLIA Waived.

Product Name	Product No.
Hemocult Mailing Pouches	62200
• 100 Mailing Pouches/box	
Hemocult ICT Collection Cards (case of 10 boxes)	395065
Each box contains:	
• 100 Collection Cards	• 1 Physician Instructions
• 100 Applicator Sticks	• 1 Sample Collection Instructions
Hemocult ICT Patient Screening Kits, 3 Day (case of 4 boxes)	395066
Each box contains:	
• 40 Patient Screening Kits, 3 Day	
• 1 Physician Instructions	
Each 3 Day Patient Screening Kit contains:	
1 Dispensing Envelope with Patient Instructions	
3 Hemocult ICT Collection Cards	
3 Flushable Collection Tissues	
3 Applicator Sticks	
1 Mailing Pouch	
Hemocult ICT Patient Screening Kits, 2 Day (case of 4 boxes)	395261
Each box contains:	
• 50 Patient Screening Kits, 2 Day	
• 1 Physician Instructions	
Each 2 Day Patient Screening Kit contains:	
1 Dispensing Envelope with Patient Instructions	
2 Hemocult ICT Collection Cards	
2 Flushable Collection Tissues	
2 Applicator Sticks	
1 Mailing Pouch	
Hemocult ICT Tests (case of 4 boxes)	395067
Each box contains:	
• 20 Test Devices	
• 1 bottle Hemocult ICT Buffer/8.0 mL	
• 1 Product Instructions	
Hemocult ICT Controls (case of 4 boxes)	395068
Each box contains:	
• 4 bottles (2 Positive and 2 Negative/0.8mL each)	
• 1 Controls Product Instructions	

Beckman Coulter, Inc.
250 S. Kraemer Blvd., Brea, CA 92821 USA

Printed in USA

©2015 Beckman Coulter, Inc.


B55355.AB
March 2015



Safe...Simple...Life-Saving

Hemocult ICT

Immunochemical Fecal Occult Blood Test

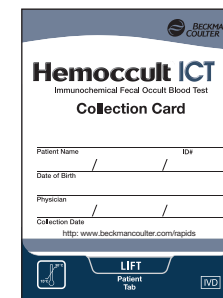

Collection Card

PATIENT SAMPLE COLLECTION INSTRUCTIONS

Sample collection for each bowel movement

